**Blom-Singer®**

voice restoration systems

R 1  
37303-01D

## Silicone Adhesive / Silikonkleber / Adhesivo de silicona / Adhésif silicone / Colla siliconica / Siliconenhechtmiddel / Adesivo de silicone

**Preparation For Use:** Remove the tamper evident seal and lid from the bottle. Retain the original cap for use when packing the bottled adhesive for transport, i.e., travel, flights, etc. Place the enclosed lid with brush on the bottle and close cap securely to minimize evaporation. A spare brush has been provided to replace the used brush, should it become dry or unusable. The used brush may be unscrewed from the cap and discarded and the replacement brush screwed in its place.

**Directions For Use:** Clean area of skin to be coated with adhesive. Remove excess adhesive from the brush before applying a very light coating to the skin surrounding the tracheostoma. Allow adhesive to dry for at least 3 to 4 minutes before attaching the flexible circular housing. For complete details of valve application, please consult your Blom-Singer Tracheostoma Valve User Information Data Sheet.

**Ingredients:** Dimethylpolysiloxane and Dichloropentafluoropropane.

**Warning: Use in a well ventilated space; do not inhale or permit adhesive to enter stoma.** If you should experience skin irritation or other discomfort around the tracheostoma area, discontinue use of this product and consult your physician. **Keep out of reach of children.** Close cap securely to minimize evaporation.

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Blom-Singer is a registered trademark of Hansa Medical Products, Inc., InHealth Technologies is a registered trademark of Helix Medical, LLC, © 2009 InHealth Technologies / Blom-Singer ist ein eingetragenes Warenzeichen der Hansa Medical Products Inc., InHealth Technologies ist ein eingetragenes Warenzeichen von Helix Medical, LLC / Blom-Singer es una marca comercial registrada de Hansa Medical Products, Inc., InHealth Technologies es una marca comercial registrada de Helix Medical, LLC, / Blom-Singer est une marque déposée de Hansa Medical Products, Inc., InHealth Technologies est une marque déposée de Helix Medical, LLC, / Blom-Singer è un marchio depositato di Hansa Medical Products, Inc., InHealth Technologies è un marchio depositato di Helix Medical, LLC, / Blom-Singer is een gedeponeerd handelsmerk van Hansa Medical Products, Inc., InHealth Technologies is een gedeponeerd handelsmerk van Helix Medical, LLC, / Blom-Singer é uma marca registrada da Hansa Medical Products, Inc., InHealth Technologies é uma marca registrada da da Helix Medical, LLC.
A Division of Helix Medical, LLC