

Support Every Step of the Way

Connect with our patient consultants and clinical education team by emailing education@inhealth.com.

We offer you support to get you started and we support you every step of the way!

Ordering is easy!

Telephone: toll-free 800.477.5969

Email: order@inhealth.com

Visit our webstore at www.inhealth.com

INHEALTH
TECHNOLOGIES®

Stay Connected with Us



United States
1110 Mark Ave.,
Carpinteria, CA 93013
Toll-Free: 800.477.5969
Toll-Free Fax: 888.371.1530

International
Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: order@inhealth.com

©2023 InHealth Technologies®—Manufactured by Freudenberg Medical, LLC (230802.00)

INHEALTH
TECHNOLOGIES®



Day&Night® HME


Let's Get Started!



Day&Night® HME

It's our newest innovation that you can wear comfortably all day and all night – including bedtime. **Love the simplicity. Love the innovation!**

The Day&Night HME cartridge contains a foam filter for heat and moisture exchange, side airflow vents to maximize airflow, and a locking mechanism to secure the device in the open position to prevent occlusion as desired.

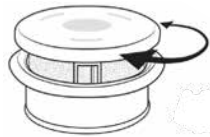

STEP 1:

Connect Day&Night to Your Blom-Singer® Adhesive Housings

Connect the Day&Night HME to your Blom-Singer Adhesive Housing. To insert the device, hold its outer rim and insert it partially into the lower rim of the previously applied tracheostoma housing. Gently push the device into the applied attachment until it snaps into place. A properly inserted HME cartridge will sit flush and fit snugly inside the housing.



STEP 2:

Test Digital Occlusion

Push with your finger or thumb against the face of the device to talk. Enjoy your HME.



STEP 3:

And at Night...

You can "lock" at night for peace of mind.



To lock: hold the tracheostoma housing or tube in place with one hand while gently turning the upper lip of the device clockwise with the other hand.

To unlock: hold the tracheostoma housing or tube in place with one hand while gently turning the upper lip of the device counterclockwise with the other hand.

