# Renú®

### VOICE

**Calcium Hydroxylapatite (CaHA)** for long term management of vocal fold insufficiency

RENÚ Voice provides greater value and cost-effectiveness compared to other injectables.<sup>1</sup>

### The power of voice!

Distributed by INHEALTH TECHNOLOGIES®

## Your Lower Cost, Higher Volume Option



#### The choice for long augmentation

#### RENÚ Voice is for long-term augmentation:

• A buffered hydrogel with synthetic, proven-biocompatible calcium hydroxylapatite (CaHA) particles 25 to 45 microns in size for consistent and easy injection.

#### Higher volume, less waste

RENÚ Voice provides a greater volume of material than competitive products: 1.5 cc vs. 1.0 cc.<sup>1</sup> The greater volume:

- Helps eliminate waste and inconvenience.
- Reduces the added cost of opening additional syringes.

## Convenience and comfort at your fingertips

RENÚ Voice is provided in sterile, ready-to-use 1.5 cc syringes with ergonomic finger grips and a 2-year shelf life.

Inject RENÚ Voice percutaneously with a needle of your choice or with the RENÚ Double-Bevel, Trans-Oral Needle, supplied separately.<sup>2</sup>

#### **INJECTION OPTIONS:**

Procedures completed by a healthcare professional

#### Operating Room (OR) Trans-Oral Injection

 Visualization via direct laryngoscopy or micro-laryngoscopy



Operating Room Trans-Oral Injection

#### Office Based Trans-Oral Injection

- Topical anesthesia
- Attach appropriate needle to RENÚ
- Prime needle with RENÚ (dead volume equals ≈0.28cc)



Office Based Trans-Oral Calcium Hydroxylapatite (CaHA)

#### **Office Based Percutaneous Injections**



Cricothyroid Cartilage approach



Thyroid Cartilage approach



Thyroid approach

## **RENÚ**: VOICE

**RENU Voice** is FDA-cleared and CE-marked for injectable VFI augmentation applications.

#### Support from a trusted global leader

RENÚ Voice is backed with comprehensive support from InHealth Technologies, a global leader.

REF	Description	Rx
08-015-04-V04	RENÚ Voice Calcium Hydroxylapatite Implant, 1.5cc	
08-015-04-VOI	RENÚ Voice Calcium Hydroxylapatite Implant, 1.5cc, CE	
12-000-00-ND1	RENÚ Transoral Needle, 24 gauge needle tip, 9.8 inches long, 16 gauge shaft	

These products are restricted to sale or use by a licensed professional only.

## For more information or to order RENÚ Voice, call InHealth Technologies at 800.477.5969, or visit www.inhealth.com.

References:

1. Data on file, InHealth Technologies. 2. Carroll TL, Rosen CA. Long-term results of calcium hydroxylapatite for vocal fold augmentation. *The Laryngoscope*. 2011;121:313–319.

#### Important safety information

RENÚ® Voice is indicated for vocal fold medialization and vocal fold insufficiency that may be improved by injection of a soft tissue-bulking agent. RENÚ® Voice injection augments the size of the displaced or deformed vocal fold so that it may meet the opposing vocal fold at the midline for improved phonation. Vocal fold insufficiency associated with serious aspiration difficulty may be an urgent indication.

After injection of RENÚ® Voice, patients may experience transient throat soreness, bruising, swelling, or other local side effects. More rare side effects may include over-injection or under-injection that may require repeated intervention. As with any injection, there may be a risk of infection.

Distributed by

**TECHNOLOGIES®** 

UNITED STATES 1110 Mark Avenue Carpinteria, CA 93013-2918 USA Toll Free: 800.477.5969 Toll Free Fax: 888.371.1530

RENÚ® is a registered trademark of Cytophil, Inc. © 2021 InHealth Technologies (201101.00) ORDER ONLINE www.inhealth.com