INHEALTH

TECHNOLOGIES®

Blom-Singer® Manometer with Manometer Adaptor

INSTRUCTIONS FOR USE

Blom-Singer®

voice restoration systems

Blom-Singer® and InHealth Technologies® are registered trademarks of **Freudenberg Medical, LLC**.

Patent(s): www.inhealthpatents.com

IFU: <https://inhealth.com/ifu>

Copyright © 2020 Freudenberg Medical, LLC. All rights reserved.

INSTRUCTIONS FOR USE

使用说明 / 使用說明書 / Udhëzime për përdorim / إرشادات الاستخدام / Инструкции за употреба / Upute za korištenje / Návod k použití / Brugsanvisning / Gebruiksaanwijzing / Kasutusjuhend / Käyttöohjeet / Instructions d'utilisation / Gebrauchsanweisung / Οδηγίες χρήσης / הוראות שימוש / Használati utasítás / Notkunarleiðbeiningar / Istruzioni per l'uso / 사용 지침 / Lietošanas norādījumi / Naudojimo instrukcijos / Bruksanvisning / Instrukcja użytkowania / Instruções de uso / Instrucțiuni de utilizare / Инструкции по применению / Uputstvo za upotrebu / Návod na použitie / Instrucciones de uso / Bruksanvisningar / Kullanım Talimatlar

DIAGRAMS

插图 / 図 / Diagramet / الرسم التوضيحي / Схеми / Dijagrami / Schémata / Diagrammer / Afbeeldingen / Diagrammid / Kaaviot / Diagrammes / Abbildungen / Διαγράμματα / תרשימים / Ábrák / Skýringarmyndir / Diagrammi / 그림 / Diagrammas / Diagramos / Diagrammer / Schematy / Diagramas / Diagrame / Схемы / Crteži / Grafické znázornenia / Diagramas / Diagram / Diyagramlar

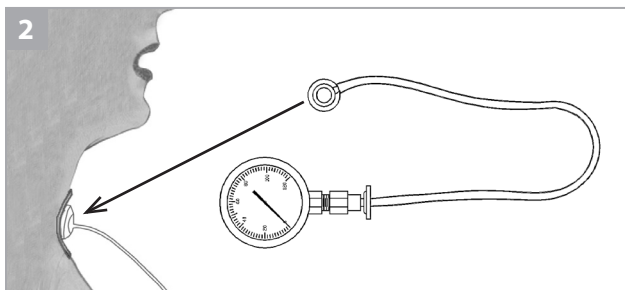
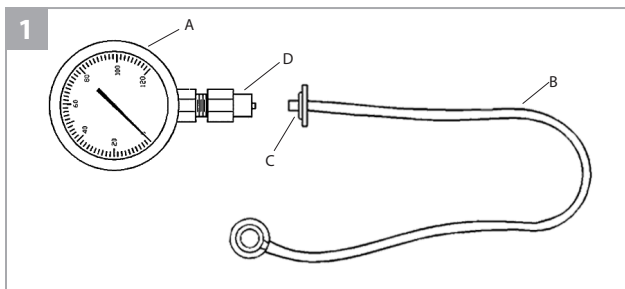
TABLE OF CONTENTS

目录 / 目次 / Tabela e përmbajtjes / جدول المحتويات / Съдържание / Sadržaj / Obsah /
 Indholdsfortegnelse / Inhoud / Sisukord / Sisällysluettelo / Table des matières /
 Inhaltsverzeichnis / Πίνακας περιεχομένων / תוכן עניינים / Tartalomjegyzék / Efnisyfirlit /
 Sommario / 목차 / Saturs / Turinys / Innholdsfortegnelse / Spis treści / Índice / Cuprins /
 Содержание / Sadržaj / Obsah / Índice / Innehållsförteckning / İçindekiler

EN	Manometer With Manometer Adaptor	5	HU	Manométer manométeradapterrel	37
ZH	附带压力表适配器的压力表	7	IS	Þrýstingsmælir ásamt millistykki fyrir þrýstingsmæli	39
JA	マノメーターアダプター付きマノメーター	8	IT	Manometro con adattatore per manometro	41
SQ	Manometër me përshtatës manometri	11	KO	압력계 어댑터를 포함하는 압력계	43
13	جهاز مقياس الضغط المزود بموانم مقياس الضغط	AR	LV	Manometrs ar manometra adapteru	45
BG	Манометър с адаптер за манометър	15	LT	Manometras su manometro adapteriu	47
HR	Manometar s manometarskim prilagodnikom	17	NO	Manometer med manometeradapter	49
CS	Manometr s adaptérem manometru	19	PL	Manometr ze złączem manometru	51
DA	Manometer med manometeradapter	21	PT	Manômetro com Adaptador de manômetro	53
NL	Manometer met manometeradapter	23	RO	Manometru cu adaptor pentru manometru	55
ET	Manomeeter manomeetri adapteriga	25	RU	Манометр с адаптером манометра	57
FI	Manometri ja manometrisovitin	27	SR	Manometar sa adapterom za manometar	59
FR	Manomètre avec adaptateur pour manomètre	29	SK	Manometer s adaptérom manometra	61
DE	Manometer mit Manometeradapter	31	ES	Manómetro con adaptador para el manómetro	63
EL	Μανόμετρο με προσαρμογέα μανόμετρου	33	SV	Manometer med manometeradapter	65
35	מד-לחץ, עם מתאם למד-לחץ (מנומטר)	HE	TR	Manometre ve Manometre Adaptörü	67

ENGLISH

BLOM-SINGER® MANOMETER WITH MANOMETER ADAPTOR**PRODUCT DESCRIPTION**

The manometer (with manometer adaptor) is supplied nonsterile in a kit containing one manometer, one manometer adaptor and one Blom-Singer Adhesive Housing. Replacement manometer adaptors and housings are available separately. The manometer is reusable between patients. The manometer adaptor and housing are disposable and intended for single use only.

Manometer with Adaptor	Manometer	Adaptor	AccuFit Housing
Manometer Kit	•	•	•
Replacement Manometer Adaptor Pack		•	•

INDICATIONS (Reasons to prescribe the device or procedure)

The manometer (diagram 1A) (with manometer adaptor) is for use by qualified, trained clinicians as an aid to teach laryngectomized patients how to self-regulate intratracheal pressure during tracheoesophageal speech.

CONTRAINDICATIONS (Reasons that make it inadvisable to prescribe the device or procedure)

The manometer (with manometer adaptor) is only indicated for laryngectomized patients who speak using a tracheoesophageal voice prosthesis.

WARNINGS AND PRECAUTIONS

If there are tears, cracks, or structural damage, do not continue to use the product. The manometer adaptor and housing are disposable and intended for single use only; reuse may result in cross contamination. The manometer is pre-calibrated at the manufacturing facility. Tampering with the instrument (e.g., unscrewing the device) may result in malfunction, inaccurate pressure readings, and/or leaks of the gauge. Tampering with the connection fittings on the manometer adaptor may result in inaccurate pressure readings.

INSTRUCTIONS FOR USE

Please refer to the diagrams located at the front of this instruction manual.

The clinician's hands should be clean and dry to help avoid introducing contaminants resulting in patient infection. The included Blom-Singer adhesive housing must first be attached to the skin surrounding the patient's tracheostoma. Follow the supplied Instructions for Use manual for instructions on application of adhesive housing. The manometer adaptor (diagram 1B) should be connected to the manometer before attaching the manometer adaptor to the tracheostoma adhesive housing.

Manometer Adaptor Attachment

1. Attach the manometer adaptor to the manometer by inserting its female luer lock connection (diagram 1C) into the male luer lock connection (diagram 1D) of the manometer. Twist the two components together until securely connected.

2. Attach the manometer adaptor to the patient by inserting the tracheostoma connector end to the tracheostoma adhesive housing (diagram 2).

Intratracheal Speech Pressure Assessment Procedure

The following procedure is provided by Eric D. Blom, Ph.D.

With the manometer adaptor inserted into the attached tracheostoma adhesive housing, instruct the patient to take a breath, occlude the adaptor with a finger, and count to 15 with normal conversational speech effort. Optimal levels are designated by color markings on the manometer gauge (see chart below). Higher levels may contribute to suboptimal voice quality, increased fatigue with voicing, and shorter duration of the tracheostoma adhesive housing seal.

Pressure Ranges (cm of H ₂ O)	
10 — 25 cm H ₂ O	Excellent — Green
26 — 40 cm H ₂ O	Good — Blue
41 — 120 cm H ₂ O	High

For some patients, observing the manometer as they vary their expiratory effort from optimal levels (usually registering 10 – 15 cm H₂O) to high (often measuring 90 cm H₂O or more) gives them feedback to monitor their expiratory effort when voicing.

Manometer Care

The manometer device is reusable and its external components may be wiped with an alcohol wipe or other similar wipe as needed.

SPECIAL STORAGE AND/OR HANDLING CONDITIONS

There are no special storage and/or handling conditions for this device.

DISPOSAL INSTRUCTIONS

This device is not biodegradable and can be contaminated when used. Carefully dispose of the device per local guidelines.

ORDERING INFORMATION

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: order@inhealth.com ORDER ONLINE: www.inhealth.com POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service INTERNATIONAL

Please contact our customer service department for a distributor referral.

PRODUCT COMPLAINTS/EU SERIOUS INCIDENTS

If you are dissatisfied with the device or have any questions, please contact productcomplaints@inhealth.com.

Telephone: +1-800-477-5969

Fax: +1-888-371-1530

Any serious incident that has occurred in relation to the device should be reported to Freudenberg Medical, LLC as noted above and the competent authority of the EU Member State in which the user and/or patient is established.

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SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Batch code; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	EN ISO 15223-1:2016	Use-by date; 5.1.4	Indicates the date after which the medical device is not to be used
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the packaged has been damaged or opened
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	EN ISO 15223-1:2016	Do not re-use; 5.4.2	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure

INHEALTH

TECHNOLOGIES®

UNITED STATES

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
Toll-Free: 800.477.5969
Toll-Free Fax: 888.371.1530

INTERNATIONAL

Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: order@inhealth.com

ORDER ONLINE

www.inhealth.com

Freudenberg Medical, LLC
1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
www.inhealth.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands