INHEALTH

TECHNOLOGIES®

Blom-Singer® Voice Prosthesis Cleaning System

INSTRUCTIONS FOR USE

Blom-Singer®

voice restoration systems

Blom-Singer® and InHealth Technologies® are registered trademarks of **Freudenberg Medical, LLC**.

Patent(s): www.inhealthpatents.com

IFU: <https://inhealth.com/ifu>

Copyright © 2020 Freudenberg Medical, LLC. All rights reserved.

INSTRUCTIONS FOR USE

使用说明 / 使用說明書 / Udhëzime për përdorim / إرشادات الاستخدام / Инструкции за употреба / Upute za korištenje / Návod k použití / Brugsanvisning / Gebruiksaanwijzing / Kasutusjuhend / Käyttöohjeet / Instructions d'utilisation / Gebrauchsanweisung / Οδηγίες χρήσης / הוראות שימוש / Használati utasítás / Notkunarleiðbeiningar / Istruzioni per l'uso / 사용 지침 / Lietošanas norādījumi / Naudojimo instrukcijos / Bruksanvisning / Instrukcja użytkowania / Instruções de uso / Instrucțiuni de utilizare / Инструкции по применению / Uputstvo za upotrebu / Návod na použitie / Instrucciones de uso / Bruksanvisningar / Kullanım Talimatlar

DIAGRAMS

插图 / 図 / Diagramet / الرسم التوضيحي / Схеми / Dijagrami / Schémata / Diagrammer / Afbeeldingen / Diagrammid / Kaaviot / Schémas / Abbildungen / Διαγράμματα / תרשימים / Ábrák / Skýringarmyndir / Diagrammi / 그림 / Diagrammas / Diagramos / Diagrammer / Schematy / Diagramas / Diagrame / Схеми / Crteži / Grafické znázornenia / Diagramas / Diagram / Diyagramlar

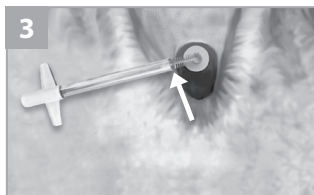
TABLE OF CONTENTS

目録 / 目次 / Tabela e përmbajtjes / جدول المحتويات / Съдържание / Sadržaj / Obsah /
 Indholdsfortegnelse / Inhoud / Sisukord / Sisällysluettelo / Table des matières /
 Inhaltsverzeichnis / Πίνακας περιεχομένων / תוכן עניינים / Tartalomjegyzék / Efnisyfirlit /
 Sommario / 목차 / Saturs / Turinys / Innholdsfortegnelse / Spis treści / Índice / Cuprins /
 Содержание / Sadržaj / Obsah / Índice / Innehållsförteckning / İçindekiler

EN	Voice Prosthesis Cleaning System	6	HU	Hangprotézis-tisztító rendszer	55
ZH	发音假体清洁系统	9	IS	Hreinsunarkerfi fyrir raddventil	58
JA	ボイスプロテーゼ洗浄システム	11	IT	Sistema per la pulizia della protesi fonatoria	61
SQ	Sistemi i pastrimit të protezës së zërit	14	KO	인공후두 세척 시스템	65
17	نظام تنظيف جهاز الحنجرة الهوائية	AR	LV	Balss protēzes tīrīšanas sistēma	68
BG	Система за почистване на протези за глас	20	LT	Balso protezo valymo sistema	71
HR	Sustav za čišćenje glasovne proteze	23	NO	Rengjøringsystem for taleprotese	74
CS	Systém pro čištění hlasové protézy	27	PL	System czyszczenia protez głosowych	77
DA	Rengøringsystem til stemmeprotese	30	PT	Sistema de limpeza da prótese vocal	80
NL	Reinigingsysteem voor stemprothese	33	RO	Sistem de curățare a protezei fonatorii	83
ET	Hääleproteesi puhastussüsteem	36	RU	Система очистки голосового протеза	87
FI	Puheproteesin puhdistusjärjestelmä	39	SR	Sistem za čišćenje glasovne proteze	90
FR	Système de nettoyage de la prothèse vocale	42	SK	Systém na čistenie hlasovej protézy	93
DE	Stimmprothesen-Reinigungssystem	45	ES	Sistema de limpieza para prótesis fonatorias	96
EL	Σύστημα καθαρισμού φωνητικής πρόθεσης	49	SV	Rengöringsystem för röstprotes	100
53	מערכת לניקוי תותב דיבור	HE	TR	Ses Protezi Temizleme Sistemi	103

ENGLISH

BLOM-SINGER® VOICE PROSTHESIS CLEANING SYSTEM

Please refer to the diagrams located at the front of this instruction manual.

PRODUCT DESCRIPTION

The Blom-Singer Voice Prosthesis Cleaning System refers to the Blom-Singer Flushing Device and the Blom-Singer Cleaning Brush. The cleaning system is meant for use by people who have a Blom-Singer voice prosthesis. The flushing device can be used with both indwelling and patient-changeable voice prostheses. The cleaning brush is only for indwelling voice prostheses. The cleaning brush is available in three sizes, corresponding to various lengths of voice prostheses (VPs): small brush (for 4mm-8mm VPs), medium brush (for 9mm-16mm VPs), and large brush (for 18mm-22mm VPs).

INDICATIONS (Reasons to prescribe a device or perform a procedure)

The Blom-Singer Flushing Device is intended to be used to flush materials that cause blockage of the lumen of either an indwelling or patient changeable Blom-Singer voice prosthesis while it is positioned in the patient's tracheoesophageal puncture (TEP; opening made by surgeon between trachea or "windpipe" and esophagus for placement of a voice prosthesis). The Blom-Singer Cleaning Brush is intended to be used to remove materials that cause blockage of the lumen of an indwelling Blom-Singer voice prosthesis while it is positioned in the patient's tracheoesophageal puncture. Blockage of the lumen of a voice prosthesis will impair the patient's ability to voice.

CONTRAINDICATIONS (Reasons that make it inadvisable to prescribe a particular device, procedure or treatment)

The cleaning system is a medical device that should be used only by individuals with training in its use and care.

Blom-Singer® Indwelling Voice Prostheses	Flushing Device	Cleaning Brush
Classic, 16 & 20 Fr.	●	●
Advantage® (Hard Valve Assembly) 20 Fr.	●	●
Advantage® (Soft Valve Assembly) 16 & 20 Fr.	●	●
Dual-Valve™, 20 Fr.	●	●
Blom-Singer® Voice Prostheses (patient changeable)	Flushing Device	
Low Pressure Voice Prosthesis, 16 & 20 Fr.	●	
Duckbill Voice Prosthesis, 16 Fr.	●	

WARNINGS AND PRECAUTIONS

The qualified, trained medical professional is to instruct patients on the use and care of this device. If there are tears, cracks, or structural damage to the cleaning devices, discontinue use. The cleaning system is for use by a single patient only. Only use the correct size cleaning brush for your prosthesis. Never fill the flushing device beyond one third of the stem.

Voice Production

The voice prosthesis lumen must be kept clear of blockage to enable patients to voice.

Voice Prosthesis Dislodgement

If the voice prosthesis is accidentally dislodged from the TE puncture, immediately place a Blom-Singer puncture dilator, or a suitable device as recommended by your clinician, of the appropriate diameter, in the puncture to keep it from closing and/or leaking fluids. Contact your clinician immediately. The voice prosthesis should be reinserted within 24 hours.

Care should be exercised when inserting or removing the cleaning devices to avoid accidental displacement of the prosthesis, which could result in aspiration (inhalation) of the voice prosthesis. In the unlikely event that this should occur, you should attempt to cough the voice prosthesis out of the windpipe. Immediately seek medical attention if unsuccessful.

Never insert foreign objects into the voice prosthesis. Use only the Blom-Singer cleaning system. Inserting objects other than the cleaning system may cause dislodgment of the voice prosthesis or its components and may cause these objects to be swallowed or inhaled.

Voice Prosthesis Leakage

Always use gentle handling and pressure when cleaning the voice prosthesis to avoid deterioration of the device and damaging internal components which could cause leakage. Recurrent leakage of fluids through the voice prosthesis should be evaluated by your clinician soon after it is noticed.

COMPLICATIONS

Although rare, the following complications may occur with the use of the cleaning system and some may require medical intervention. They include: Patient discomfort or tissue irritation; The cleaning brush may pierce the tracheal or esophageal tissue causing damage; Use of a damaged device may result in the device or components being aspirated or ingested; Contaminants introduced into the tracheostoma, puncture or esophagus; Patient infection or cross-contamination; Cleaning device or voice prosthesis is damaged and cannot be used. Refer to the voice prosthesis instructions for use for voice prosthesis related complications.

VOICE PROSTHESIS CLEANING AND CARE

INSTRUCTIONS *(Please refer to the diagrams located at the front of this instruction manual)*

Your clinician will instruct you on the routine care and cleaning of the Blom-Singer voice prosthesis. The user's hands should always be washed with soap and water prior to using the cleaning system. The cleaning system provides a means for careful cleaning of the voice prosthesis while in your TE puncture. Cleaning of the voice prosthesis should only be done in front of a mirror with a bright light focused directly on the stoma so that the open end of the prosthesis is clearly visible. Only air dry or use a lint free cloth or tissue to dry the device. Use of non-lint free materials may leave debris which can be aspirated into the airway. Do not use solvents or petroleum-based products for cleaning or lubricating the device. These materials may damage silicone or cause the device to not work properly.

Cleaning Brush

Prior to using the cleaning brush (**diagram 1**), inspect it for any structural damage; do not use a device that is damaged. Moisten the cleaning brush in clean water before use. Carefully and gently insert the cleaning brush fully into the indwelling voice prosthesis (**diagram 3**). Brush should only be inserted up to safety stop where the bristles end (**diagram 3**). Never force the cleaning brush through the voice prosthesis. Gently rotate the cleaning brush clockwise and counterclockwise multiple times to clear the voice prosthesis lumen of blockage. After brushing, remove the cleaning brush carefully. If the cleaning brush will not readily rotate, the voice prosthesis lumen may be blocked and should therefore be flushed with the flushing device to loosen the materials that

cause blockage (refer to flushing device instructions). Verify that blockage of the lumen has been removed by voicing.

If unable to voice, repeat the cleaning procedure. If still unable to voice, contact your clinician. After use, the cleaning brush should be inspected and any visible debris should be removed. Remove visible debris by holding the brush under a running stream of water. After visible debris is removed, disinfect the cleaning brush by immersion into a solution of 3% hydrogen peroxide for 1 hour.

Flushing Device

To fill the flushing device (**diagram 2**) with water, squeeze the bulb and then release it in water. Shake excess water from the tip of the flushing device before flushing the voice prosthesis. Carefully and gently insert the tip of the flushing device into the voice prosthesis until it fits completely against the opening of the prosthesis (**diagram 4**). (Note: Do not flush water through the voice prosthesis until the tip of the flushing device is properly inserted into the prosthesis, thus avoiding leaking water into the trachea. Should a small amount of water leak into the trachea, this will cause coughing). Squeeze the bulb on the flushing device to flush a rapid squirt of water through the voice prosthesis. After flushing, remove the flushing device carefully. If water will not readily squirt through the voice prosthesis, this indicates that the voice prosthesis lumen may be blocked. Allow a few drops of water to loosen the materials that cause blockage for a few minutes and then flush again. Verify that blockage of the lumen has been removed by voicing.

If unable to voice, repeat the cleaning procedure. If still unable to voice and using an indwelling voice prosthesis use the cleaning brush as instructed above. If using a patient changeable voice prosthesis, remove the voice prosthesis device, wash with a mild detergent, and rinse under a strong flush of tap water. After use, the flushing device should be inspected and any visible debris should be removed. Remove visible debris by holding the flushing device under a running stream of water. After visible debris is removed, disinfect the flushing device by filling the device and immersing it into a solution of 3% hydrogen peroxide for 1 hour.

SPECIAL STORAGE AND/OR HANDLING CONDITIONS

Only the flushing device has any special storage and/or handling conditions. It must NOT be stored in a dark, non-aerated area.

DISPOSAL INSTRUCTIONS

This product is not biodegradable and must be considered contaminated upon use. Carefully dispose of this device per local guidelines.

ORDERING INFORMATION

USA

Blom-Singer® products may be ordered directly from InHealth Technologies. **TELEPHONE:** Toll-Free (800)477-5969 or (805)684-9337, Monday — Friday, 9:30 am — 7:00 pm, Eastern Standard Time.

FAX: Toll-Free (888)371-1530 or (805)684-8594. **EMAIL:** order@inhealth.com. **ORDER ON-LINE:** www.inhealth.com. **POST:** InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service.

INTERNATIONAL

Please contact our customer service department for a distributor referral.

Product Complaints/EU Serious Incidents

If you are dissatisfied with the device or have any questions, please contact productcomplaints@inhealth.com

Telephone: +1-800-477-5969 Fax: +1-888-371-1530

Any serious incident that has occurred in relation to the device should be reported to Freudenberg Medical, LLC as noted above and the competent authority of the EU Member State in which the user and/or patient is established.

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SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Batch code; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	EN ISO 15223-1:2016	Use-by date; 5.1.4	Indicates the date after which the medical device is not to be used
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the packaged has been damaged or opened
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	ISO 7000	Packaging unit; 2794	To indicate the number of pieces in the package

INHEALTH

TECHNOLOGIES®

UNITED STATES

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
Toll-Free: 800.477.5969
Toll-Free Fax: 888.371.1530

INTERNATIONAL

Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: order@inhealth.com

ORDER ONLINE

www.inhealth.com

Freudenberg Medical, LLC
1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
www.inhealth.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands