INHEALTH

TECHNOLOGIES®

Katz® Extractor Oto-Rhino Foreign Body Remover

MEDICAL PROFESSIONAL INSTRUCTIONS FOR USE

Blom-Singer® and InHealth Technologies® are registered trademarks of Freudenberg Medical, LLC.

Patent(s): www.inhealthpatents.com

IFU: <https://inhealth.com/ifu>





Copyright © 2020 Freudenberg Medical, LLC. All rights reserved.

MEDICAL PROFESSIONAL INSTRUCTIONS FOR USE

医学专业使用说明 / 医療専門家用取扱説明書 / Udhëzimet e përdorimit për mjekët profesionistë / إرشادات الاستخدام للمتخصصين في المجال الطبي / Инструкции за употреба за медицински специалисти / Upute za korištenje za zdravstveno osoblje / Návod k použití pro zdravotníky / Brugsanvisning til uddannede læger / Gebruiksaanwijzing voor de medische professional / Tervishoiutöötaja kasutusjuhend / Lääketieteen ammattilainen käyttöohjeet / Instructions d'utilisation destinées aux professionnels de santé / Gebrauchsanweisung für medizinische Fachkräfte / Οδηγίες χρήσης για τον επαγγελματία υγείας / הוראות שימוש לצוות הרפואי / Egészségügyi szakembereknek szóló használati utasítás / Notkunarleiðbeiningar fyrir heilbrigðisstarfsmenn / Istruzioni per l'uso destinate al personale sanitario / 의료 전문가 사용 지침 / Medicinas speciālistam paredzētas lietošanas instrukcijas / Medikams profesionalams skirta naudojimo instrukcija / Bruksanvisning for medisinske fagpersoner / Instrukcja obsługi dla personelu medycznego / Instruções de uso para profissionais médicos / Instrucțiuni de utilizare pentru cadre medicale specializate / Инструкции по применению для профессиональных медработников / Uputstvo za upotrebu za zdravstvenog radnika / Návod na použitie pre zdravotníckych pracovníkov / Instrucciones de uso para profesionales médicos / Sjukvårdspersonal, bruksanvisning / Sağlık Uzmanı Kullanım Talimatları

DIAGRAMS

插图 / 図 / Diagramet / الرسم التوضيحي / Схеми / Dijagrami / Schémata / Diagrammer /
 Afbeeldingen / Diagrammid / Kaaviot / Diagrammes / Abbildungen / Διαγράμματα / תרשימים /
 Ábrák / Skýringarmyndir / Diagrammi / 그림 / Diagrammas / Diagramos / Diagrammer /
 Schematy / Diagramas / Diagrame / Схеми / Crteži / Grafické znázornenia / Diagramas /
 Diagram / Diyagramlar

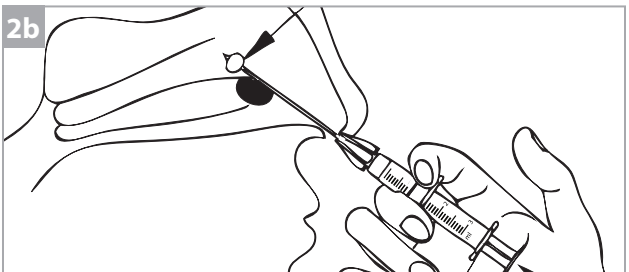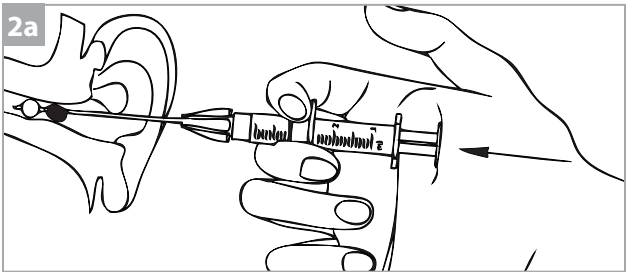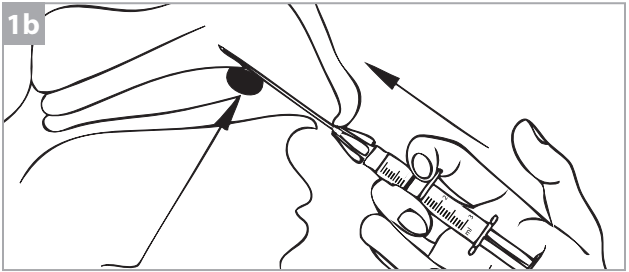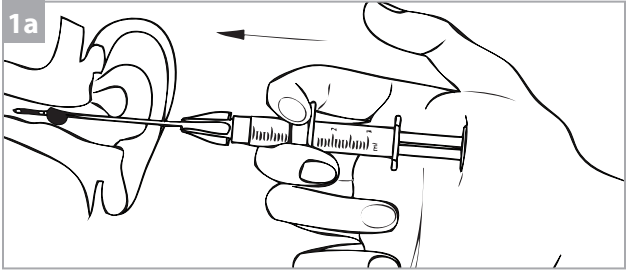
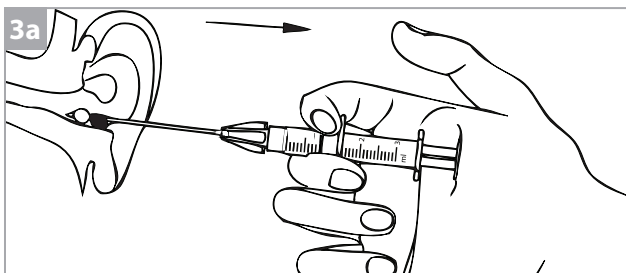
TABLE OF CONTENTS

目録 / 目次 / Tabela e përmbajtjes / جدول المحتويات / Съдържание / Sadržaj / Obsah /
 Indholdsfortegnelse / Inhoud / Sisukord / Sisällysluettelo / Table des matières /
 Inhaltsverzeichnis / Πίνακας περιεχομένων / תוכן עניינים / Tartalomjegyzék / Efnisyfirlit /
 Sommario / 목차 / Saturs / Turinys / Innholdsfortegnelse / Spis treści / Índice / Cuprins /
 Содержание / Sadržaj / Obsah / Índice / Innehållsförteckning / İçindekiler

EN	Katz Extractor	6	HU	Katz extraktor	24
ZH	Katz 提取器	7	IS	Katz útdráttaráhald	25
JA	Katz エクストラクタ	8	IT	Katz Extractor	27
SQ	Ekstraktori "Katz"	9	KO	Katz Extractor	28
10	Katz Extractor	AR	LV	Katz Extractor	29
BG	Katz Extractor	11	LT	Ekstraktori „Katz“	30
HR	Katzov ekstraktor	12	NO	Katz Extractor-anordning	31
CS	Extraktor Katz	13	PL	Ekstraktor Katz	32
DA	Katz-ekstraktor	15	PT	Extrator Katz	34
NL	Katz Extractor	16	RO	Extractor Katz	35
ET	Katz Extractor	17	RU	Экстрактoр Katz	36
FI	Katz-poistoväline	18	SR	Katz ekstraktor	37
FR	Extracteur Katz	19	SK	Extraktor Katz	38
DE	Katz Extraktor	21	ES	Katz Extractor	40
EL	Εξαγωγέας Katz	22	SV	Katz extraktor	41
23	Katz Extractor אביזר כץ	HE	TR	Katz Ekstraktör	42

KATZ® EXTRACTOR Oto-Rhino Foreign Body Remover

Developed in conjunction with David L. Katz, MD, MPH

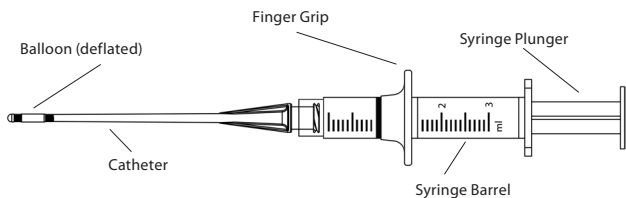**PRODUCT DESCRIPTION**

The Katz Extractor is supplied nonsterile; do not sterilize as damage to the device may occur. It is available in a convenient, ready-to-use, five-pack dispenser box.

INDICATIONS (Reasons to prescribe the device or procedure)

The Katz Extractor oto-rhino foreign body remover is designed and intended principally for extraction of impacted foreign bodies in the nasal passages. This device may also be used in the external ear canal, based upon clinical judgment.

CONTRAINDICATIONS (Reasons that make it inadvisable to prescribe the particular device or procedure)

- Do not use if the device appears to be damaged.
- Do not use the device if bleeding is evident in the nasal or ear canal.
- Do not use the device if you suspect there is not adequate space for the balloon to inflate. Increased pain would be an indicator of this.

WARNINGS AND PRECAUTIONS

This device is intended for single use only. Reuse of the device may lead to patient infection. Use in the nasal canal or external ear canal only. Do not apply traction to a foreign body with sharp edges, as this may abrade or lacerate the tissue. This may also damage the catheter balloon and deflate it, or cause the device to get stuck. If, after inflation of the balloon, the impacted foreign body does not freely move out of the canal with minimal resistance (pull force), then deflate the balloon and remove the device. Repositioning of the catheter should be considered, as dictated by clinical judgment. Be aware that the foreign body may be lodged further into the canal. Usage of this device in the ear canal may cause injury to the tympanic membrane; use of other methods, such as flushing, should be considered and medical judgment applied. This device is designed as a single unit. Do not replace or remove the catheter from the syringe. Do not fill the syringe with anything other than air.

INSTRUCTIONS FOR USE

The Katz Extractor is for use only by qualified, trained medical professionals. Note: The maximum inflation level of 1 cc is monitored by a stop. Do not attempt to exceed the stop by use of force. The following procedural instructions are provided by David L. Katz, M.D., MPH. Please refer to the diagrams located at the front of this instruction manual.

1. Insert the catheter, with the balloon in the deflated state, along the canal until the balloon has been positioned behind the impacted foreign body. Use the attached finger grip for additional stability, diagram 1a, 1b.
2. Inflate the balloon by depressing the syringe plunger until limited resistance from the canal wall is perceived, diagram 2a, 2b.
3. Retract the catheter out of the canal with the balloon in the inflated state (plunger depressed), dislodging the foreign body in the process, diagram 3a, 3b.

SPECIAL STORAGE AND/OR HANDLING CONDITIONS

There are no special storage and/or handling conditions for this device.

DISPOSAL INSTRUCTIONS

This device is not biodegradable and can be contaminated when used. Carefully dispose of the device per local guidelines.

ORDERING INFORMATION USA

Katz® products may be ordered directly from InHealth Technologies

TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: order@inhealth.com ORDER ON-LINE: www.inhealth.com POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service

INTERNATIONAL

Please contact our customer service department for a distributor referral.


PRODUCT COMPLAINTS/EU SERIOUS INCIDENTS

If you are dissatisfied with the device or have any questions, please contact productcomplaints@inhealth.com.

Telephone: +1-800-477-5969

Fax: +1-888-371-1530

Any serious incident that has occurred in relation to the device should be reported to Freudenberg Medical, LLC as noted above and the competent authority of the EU Member State in which the user and/or patient is established.

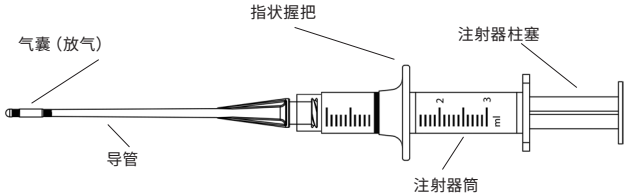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SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Batch code; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	EN ISO 15223-1:2016	Use-by date; 5.1.4	Indicates the date after which the medical device is not to be used
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the packaged has been damaged or opened
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	EN ISO 15223-1:2016	Do not re-use; 5.4.2	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure
	EN ISO 15223-1:2016	Non-sterile; 5.2.7	Indicates a medical device that has not been subjected to a sterilization process.
	ISO 7000:2019	Packaging unit; 2794	To indicate the number of pieces in the package

INHEALTH

TECHNOLOGIES®

UNITED STATES

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
Toll-Free: 800.477.5969
Toll-Free Fax: 888.371.1530

INTERNATIONAL

Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: order@inhealth.com

ORDER ONLINE

www.inhealth.com

Freudenberg Medical, LLC
1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
www.inhealth.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands