INHEALTH

TECHNOLOGIES®

Blom-Singer® Voice Prosthesis Cleaning System

INSTRUCTIONS FOR USE

Blom-Singer®

voice restoration systems

Blom-Singer® and InHealth Technologies® are registered trademarks of **Freudenberg Medical, LLC**.

IFU: <https://inhealth.com/ifu>

Copyright © 2024 Freudenberg Medical, LLC. All rights reserved.

INSTRUCTIONS FOR USE

使用说明 / 取扱説明書 / Udhëzimet E Përdorimit / إرشادات الاستخدام / Инструкции За Употреба / Upute Za Korištenje / Návod K Použití / Bruksanvisning / Gebruiksaanwijzingen / Kasutusjuhend / Käyttöohjeet / Instructions D'utilisation / Gebrauchsanweisung / Οδηγίες Χρήσης / הוראות שימוש / Használati Utasítás / Notkunarleiðbeiningar / Istruzioni Per L'uso / 사용 지침 / Lietošanas Pamācība / Naudojimo Instrukcija / Bruksanvisning / Instrukcja Użytkowania / Instruções De Uso / Instrucțiuni De Utilizare / Инструкции по применению / Uputstva za upotrebu / Návod na použitie / instrucciones de uso / Bruksanvisning / Kullanim talimatları

DIAGRAMS

插图 / 図 / Diagramet / الرسوم التوضيحية / Схеми / Dijagrami / Schémata / Diagrammer / Afbeeldingen / Diagrammid / Kaaviot / Diagrammes / Abbildungen / Διαγράμματα / תרשימים / Ábrák / Skýringarmyndir / Diagrammi / 그림 / Diagrammas / Paveikslėliai / Diagrammer / Schematy / Diagramas / Diagrame / Схемы / Dijagrami / Grafické Znážomenia / Diagramas / Diagram / Diyagramlar

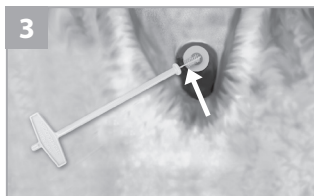
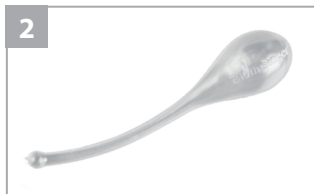
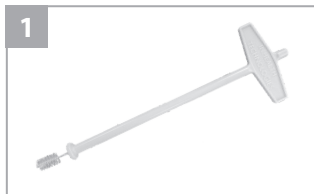
TABLE OF CONTENTS

目录 / 目次 / Tabela E Përmbajtjes / جدول المحتويات / Съдържание / Sadržaj / Obsah /
 Indholdsfortegnelse / Inhoud / Sisukord / S isällisluettelo / Table Des Matières /
 Inhaltsverzeichnis / Πίνακας Περιεχομένων / תוכן עניינים / Tartalomjegyzék / Efnisyfirlit /
 Sommario / 목차 / Saturs / Turinys / Innholdsfortegnelse / Spis Treści / Índice / Cuprins /
 Содержание / Sadržaj / Obsah / Índice / Innehållsförteckning / İçindekiler

EN	Voice Prosthesis Cleaning System	5	HU	Hangprotézis-tisztító rendszer	59
ZH	发音假体清洁系统	9	IS	Hreinsunarkerfi fyrir raddventil	62
JA	音声プロテーゼクリーニングシステム	11	IT	Sistema per la pulizia della protesi fonatoria	65
SQ	Sistemi i pastrimit të protezës së zërit	14	KO	인공후두 세척 시스템	69
18	نظام تنظيف جهاز الحنجرة الهوائية	AR	LV	Balss protēzes tīrīšanas sistēma	72
BG	Система за почистване на протези за глас	21	LT	Balso protezo valymo sistema	75
HR	Sustav za čišćenje glasovne proteze	24	NO	Rengjøringsystem for stemmeprotese	78
CS	Systém pro čištění hlasové protézy	28	PL	System czyszczenia protezy głosowej	82
DA	Rengøringsystem til stemmeprotese	31	PT	Sistema de limpeza da prótese vocal	85
NL	Reinigingsysteem voor de stemprothese	34	RO	Sistem de curățare a protezei fonatorii	89
ET	Hääleproteesi puhastussüsteem	38	RU	Система очистки голосового протеза	92
FI	Puheproteesin puhdistusjärjestelmä	41	SR	Sistem za čišćenje glasovne proteze	96
FR	Système de nettoyage de la prothèse vocale	44	SK	Systém na čistenie hlasovej protézy	99
DE	Stimmprothesen-Reinigungssystem	48	ES	Sistema de limpieza de la prótesis fonatoria	103
EL	Σύστημα καθαρισμού φωνητικής πρόθεσης	52	SV	Rengöringsystem för röstprotes	106
56	מערכת לניקוי תותב דיבור	HE	TR	Ses Protezi Temizleme Sistemi	109

ENGLISH

BLOM-SINGER® VOICE PROSTHESIS CLEANING SYSTEM

Please refer to the diagrams located at the front of this instruction manual.

PRODUCT DESCRIPTION

The Blom-Singer Voice Prosthesis Cleaning System refers to the Blom-Singer Flushing Device and the Blom-Singer Cleaning Brush. The cleaning system is meant for use by people who have a Blom-Singer voice prosthesis. The flushing device can be used with both indwelling and patient-changeable voice prostheses. The cleaning brush is only for indwelling voice prostheses. The cleaning brush is available in two sizes, corresponding to various lengths of voice prostheses (VPs): standard brush (for 4mm-10mm VPs), and long brush (for 12mm-20mm VPs).

INDICATIONS (Reasons to prescribe a device or perform a procedure)

The Blom-Singer Flushing Device is intended to be used to flush materials that cause blockage of the lumen of either an indwelling or patient changeable Blom-Singer voice prosthesis while it is positioned in the patient's tracheoesophageal puncture (TEP; opening made by surgeon between trachea or "windpipe" and esophagus for placement of a voice prosthesis). The Blom-Singer Cleaning Brush is intended to be used to remove materials that cause blockage of the lumen of an indwelling Blom-Singer voice prosthesis while it is positioned in the patient's tracheoesophageal puncture. Blockage of the lumen of a voice prosthesis will impair the patient's ability to voice.

CONTRAINDICATIONS (Reasons that make it inadvisable to prescribe a particular device, procedure or treatment)

The cleaning system is a medical device that should be used only by individuals with training in its use and care.

Voice Prosthesis Type	Flushing Device	Cleaning Brush
Blom-Singer® Indwelling Voice Prostheses - Clinician Placed (Classic, Advantage®, Dual-Valve™, Special Order, 22.5 Fr.)	●	●
Blom-Singer® Voice Prostheses - Patient Changeable (Duckbill, Low Pressure, Special Order Low Pressure)	●	

WARNINGS AND PRECAUTIONS

The qualified, trained medical professional is to instruct patients on the use and care of this device. If there are tears, cracks, or structural damage to the cleaning devices, discontinue use. The cleaning system is for use by a single patient only. Only use the correct size cleaning brush for your prosthesis. Never fill the flushing device beyond one third of the stem.

Voice Production

The voice prosthesis lumen must be kept clear of blockage to enable patients to voice.

Voice Prosthesis Dislodgement

Care must be exercised when inserting or removing the cleaning devices to avoid accidental displacement of the voice prosthesis, which could result in aspiration (inhalation) of the voice

prosthesis into the trachea (windpipe). Should aspiration occur, the patient should attempt to cough the voice prosthesis out of the trachea. Further medical attention may be necessary if coughing the device out is unsuccessful. If the voice prosthesis is dislodged from the TEP, a Blom-Singer Puncture Dilator or suitable device of the appropriate diameter should be immediately placed in the puncture to keep it from closing and leaking fluids. A replacement voice prosthesis should be inserted within 24 hours.

Foreign objects should not be inserted into the voice prosthesis. Inserting objects other than the Blom-Singer cleaning devices may cause dislodgment and subsequent aspiration or ingestion of the voice prosthesis or its components.

Voice Prosthesis Leakage

Use gentle handling and pressure when cleaning the voice prosthesis to avoid deterioration of the device and damaging internal components which could cause leakage. Recurrent leakage of the voice prosthesis should be evaluated by a clinician as leakage could cause aspiration pneumonia.

COMPLICATIONS

Although rare, the following complications may occur with the use of the cleaning system and some may require medical intervention. They include: Patient discomfort or tissue irritation; The cleaning brush may pierce the tracheal or esophageal tissue causing damage; Use of a damaged device may result in the device or components being aspirated or ingested; Contaminants introduced into the tracheostoma, puncture or esophagus; Patient infection or cross-contamination; Cleaning device or voice prosthesis is damaged and cannot be used. Refer to the voice prosthesis instructions for use for voice prosthesis related complications.

VOICE PROSTHESIS CLEANING AND CARE INSTRUCTIONS

(Please refer to the diagrams located at the front of this instruction manual)

Your clinician will instruct you on the routine care and cleaning of the Blom-Singer voice prosthesis. The user's hands should always be washed with soap and water prior to using the cleaning system. The cleaning system provides a means for careful cleaning of the voice prosthesis while in your TE puncture. Cleaning of the voice prosthesis should only be done in front of a mirror with a bright light focused directly on the stoma so that the open end of the prosthesis is clearly visible. Only air dry or use a lint free cloth or tissue to dry the device. Use of non-lint free materials may leave debris which can be aspirated into the airway. Do not use solvents or petroleum-based products for cleaning or lubricating the device. These materials may damage silicone or cause the device to not work properly.

Cleaning Brush

Prior to using the cleaning brush (**diagram 1**), inspect it for any structural damage; do not use a device that is damaged. Moisten the cleaning brush in clean water before use. Carefully and gently insert the cleaning brush fully into the center opening of the indwelling voice prosthesis (**diagram 3**). The brush may be rotated for a smoother insertion and removal. Brush should be inserted only as much as the length of your specific voice prosthesis, at a maximum to where the bristles end (**diagram 3**). Never force the cleaning brush through the voice prosthesis. Gently rotate the cleaning brush clockwise and counterclockwise multiple times to clear the voice prosthesis lumen of blockage. If the cleaning brush will not readily rotate, the voice prosthesis lumen may be blocked and should therefore be flushed with the flushing device to loosen the materials that cause blockage (refer to flushing device instructions). After brushing, remove the cleaning brush carefully. Verify that blockage of the lumen has been removed by voicing.

If unable to voice, repeat the cleaning procedure. If still unable to voice, contact your clinician. After use, the cleaning brush should be inspected and any visible debris should be removed. Remove

visible debris by holding the brush under a running stream of water. After visible debris is removed, disinfect the cleaning brush by immersion into a solution of 3% hydrogen peroxide for 1 hour.

Flushing Device

To fill the flushing device (**diagram 2**) with water, squeeze the bulb and then release it in water. Shake excess water from the tip of the flushing device before flushing the voice prosthesis. Carefully and gently insert the tip of the flushing device into the voice prosthesis until it fits completely against the opening of the prosthesis (**diagram 4**). (Note: Do not flush water through the voice prosthesis until the tip of the flushing device is properly inserted into the prosthesis, to avoid leaking water into the trachea. Should water leak into the trachea, this will cause coughing). Squeeze the bulb on the flushing device to flush water through the voice prosthesis. If water will not readily pass through the voice prosthesis, this indicates that the voice prosthesis lumen may be blocked. After flushing, remove the flushing device carefully. Verify that blockage of the lumen has been removed by voicing.

If unable to voice, repeat the cleaning procedure. If still unable to voice and using an indwelling voice prosthesis use the cleaning brush as instructed above. If using a patient-changeable voice prosthesis, the device can be removed, cleaned, and reinserted as per the IFU for your specific prosthesis. After use, the flushing device should be inspected and any visible debris should be removed. Remove visible debris by holding the flushing device under a running stream of water. After visible debris is removed, disinfect the flushing device by filling the device and immersing it into a solution of 3% hydrogen peroxide for 1 hour.

Device Replacement

The Blom Singer Cleaning Brush and the Blom Singer Flushing Device are designed for reuse by a single patient. The devices can be used as long as they are cleaned as directed, undamaged, and function to the user's satisfaction. The cleaning system must be replaced when the voice prostheses is replaced, or after 6 months of use, whichever occurs first.

SPECIAL STORAGE AND/OR HANDLING CONDITIONS

Only the flushing device has any special storage and/or handling conditions. It must be stored in a well ventilated area.

DISPOSAL INSTRUCTIONS

This product is not biodegradable and must be considered contaminated upon use. Carefully dispose of this device per local guidelines.

ORDERING INFORMATION

USA

InHealth Technologies® products may be ordered directly from Freudenberg Medical, LLC, d/b/a "InHealth Technologies."

Telephone: Toll-Free 800.477.5969 or +1.805.684.9337

Fax: Toll-Free 888.371.1530 or +1.805.684.8594

Email: order@inhealth.com Order Online: www.inhealth.com

Mail: InHealth Technologies 1110 Mark Avenue Carpinteria, CA 93013-2918 USA

INTERNATIONAL

InHealth Technologies products are available through authorized distributors worldwide. For international distributor information, please contact our customer service department.

PRODUCT COMPLAINTS/EU SERIOUS INCIDENTS

If you are dissatisfied with the device or have any questions, please contact
productcomplaints@inhealth.com

Telephone: +1.800.477.5969

Fax: +1.888.371.1530

Any serious incident that has occurred in relation to the device should be reported to Freudenberg Medical, LLC as noted above and the competent authority of the EU Member State in which the user and/or patient is established.

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SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Batch code; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	EN ISO 15223-1:2016	Use-by date; 5.1.4	Indicates the date after which the medical device is not to be used
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the packaged has been damaged or opened
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	ISO 7000	Packaging unit; 2794	To indicate the number of pieces in the package

INHEALTH

TECHNOLOGIES®

UNITED STATES

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
Toll-Free: 800.477.5969
Toll-Free Fax: 888.371.1530

INTERNATIONAL

Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: order@inhealth.com

ORDER ONLINE

www.inhealth.com

Freudenberg Medical, LLC

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
www.inhealth.com

EMERGO EUROPE

Westervoortsedijk 60
6827 AT Arnhem
The Netherlands