INHEALTH

TECHNOLOGIES®

Blom-Singer® HME/HumidiFilter® System

INSTRUCTIONS FOR USE

Blom-Singer®

voice restoration systems

37602-01H Effective January 2020 / 2020 年 1 月生效 / 2020年1月発効 / Në fuqi nga janari 2020 / 2020 اعتبارًا من شهر يناير / В сила от януари 2020 г. / Stupa na snagu u siječnju 2020. / V platnosti od ledna 2020 / Gældende fra januar 2020 / Met ingang van januari 2020 / Kehtib alates jaanuarist 2020 / Voimassa tammikuusta 2020 alkaen / Date d'entrée en vigueur : janvier 2020 / Gültig ab Januar 2020 / Σε ισχύ από τον Ιανουάριο του 2020 / 2020 בתוקף החל מינואר / Hatályos 2020. januártól / Gildir frá janúar 2020 / Valido da gennajo 2020 / 2020년 1월 발효 / Spēkā no 2020. gada janvāra / Įsigalioja 2020 m. sausio mėn. / Gjelder fra januar 2020 / Wchodzi w życie w styczniu 2020 r. / Vigente em janeiro de 2020 / Cu intrare în vigoare din ianuarie 2020 / Версия от января 2020 г. / Važi od januara 2020. / Účinné od januára 2020 / Efectivo a partir de enero de 2020 / Gäller från januari 2020 / Geçerlilik Tarihi: Ocak 2020

Blom-Singer® and InHealth Technologies® are registered trademarks of Freudenberg Medical, LLC.

Patent(s): www.inhealthpatents.com






Copyright © 2020 Freudenberg Medical, LLC. All rights reserved.

INSTRUCTIONS FOR USE

使用说明 / 使用說明書 / Udhëzime për përdorim / إرشادات الاستخدام / Инструкции за употреба / Upute za korištenje / Návod k použití / Brugsanvisning / Gebruiksaanwijzing / Kasutusjuhend / Käyttöohjeet / Instructions d'utilisation / Gebrauchsanweisung / Οδηγίες χρήσης / הוראות שימוש / Használati utasítás / Notkunarleiðbeiningar / Istruzioni per l'uso / 사용 지침 / Lietošanas norādījumi / Naudojimo instrukcijos / Bruksanvisning / Instrukcja użytkowania / Instruções de uso / Instrucțiuni de utilizare / Инструкции по применению / Uputstvo za upotrebu / Návod na použitie / Instrucciones de uso / Bruksanvisningar / Kullanım Talimatlar

DIAGRAMS

插图 / 図 / Diagramet / الرسم التوضيحي / Схеми / Dijagrami / Schémata / Diagrammer / Afbeeldingen / Diagrammid / Kaaviot / Diagrammes / Abbildungen / Διαγράμματα / תרשימים / Ábrák / Skýringarmyndir / Diagrammi / 그림 / Diagrammas / Diagramos / Diagrammer / Schematy / Diagramas / Diagrame / Схемы / Crteži / Grafické znázornenia / Diagramas / Diagram / Diyagramlar

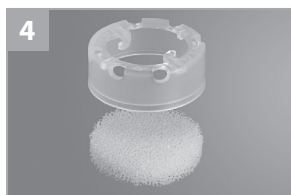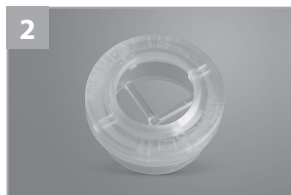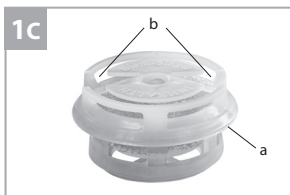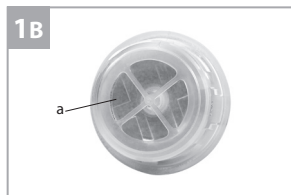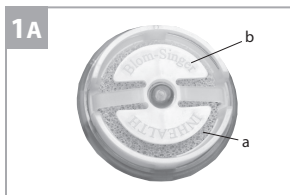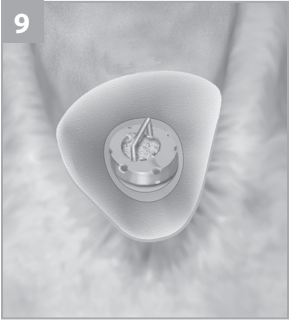
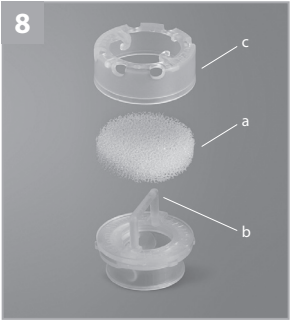
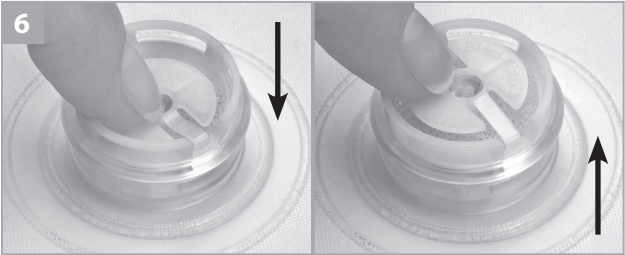
TABLE OF CONTENTS

目录 / 目次 / Tabela e përmbajtjes / جدول المحتويات / Съдържание / Sadržaj / Obsah /
 Indholdsfortegnelse / Inhoud / Sisukord / Sisällysluettelo / Table des matières /
 Inhaltsverzeichnis / Πίνακας περιεχομένων / תוכן עניינים / Tartalomjegyzék / Efnisyfirlit /
 Sommario / 목차 / Saturs / Turinys / Innholdsfortegnelse / Spis treści / Índice / Cuprins /
 Содержание / Sadržaj / Obsah / Índice / Innehållsförteckning / İçindekiler

EN	HME/HumidiFilter System	6	HU	HME/HumidiFilter rendszer	65
ZH	HME/HumidiFilter 系统	9	IS	HME/HumidiFilter-kerfi	68
JA	HME/HumidiFilter System	13	IT	Sistema HME/HumidiFilter	72
SQ	Sistemi HME/"HumidiFilter"	17	KO	HME/습도 필터 시스템	76
20	HME/HumidiFilter نظام	AR	LV	HME/HumidiFilter sistēma	79
BG	HME/HumidiFilter Система	24	LT	Sistema „HME/HumidiFilter“	83
HR	Sustav HME/HumidiFilter	28	NO	HME/HumidiFilter-system	86
CS	Systém HME/HumidiFilter	31	PL	System HME/HumidiFilter	90
DA	HME/HumidiFilter-system	35	PT	Sistema de HME/HumidiFilter	93
NL	HME/HumidiFilter-systeem	38	RO	Sistem HME/HumidiFilter	97
ET	HME/HumidiFilteri süsteem	42	RU	Система HME/HumidiFilter	101
FI	HME/HumidiFilter-järjestelmä	46	SR	HME/HumidiFilter sistem	105
FR	HME/Système HumidiFilter	49	SK	HME/HumidiFilter System	109
DE	HME/HumidiFilter System	53	ES	Sistema HME/HumidiFilter	112
EL	Σύστημα HME/HumidiFilter	57	SV	HME/HumidiFilter System	116
61	HME/HumidiFilter מערכת	HE	TR	HME/HumidiFilter Sistemi	120

ENGLISH

BLOM-SINGER® HME/HUMIDIFILTER® SYSTEM

PRODUCT DESCRIPTION

The Blom-Singer HME/HumidiFilter Systems are available in the following options:

- HME Cartridge (diagram 1) – supplied in a package of 30 HME Cartridges (EasyFlow® or ClassicFlow® options); pre-assembled, 24-hour use and single-use; contains: a HumidiFilter foam filter for heat and moisture exchange (top view, diagram 1A-a), an EasyTouch™ speech button (top view, diagram 1A-b) that is depressed to occlude the stoma for talking, a MucusShield™ occluder (bottom view, diagram 1B-a) to protect from mucus secretions, a finger-lip (side view, diagram 1C-a) for ease of removal of the HME Cartridge from the laryngectomy tube or tracheostoma/adhesive housing; front and side airflow vents to maximize airflow (side view, diagram 1C-b).
- HumidiFilter® Holder (diagram 2) and Foam Filters (diagram 3) – supplied as one holder; a lightweight plastic holder for HumidiFilter Foam Filter (filters are supplied separately); the holder and filters require assembly; the holder is reusable and filters are single-use.
- ATSV II HumidiFilter® Cap (diagram 4) and 7 Foam Filters – supplied as 1 cap and 7 foam filters, only to be paired with ATSV-II hands-free device; requires assembly; the cap is reusable and the foam filters are single-use; twist-on plastic cap holds a HumidiFilter Foam Filter and fits over the Blom-Singer Adjustable Tracheostoma Valve (ATSV II) hands-free device; replacement filters are supplied separately.

Attachment of the above HME/HumidiFilter systems to the tracheostoma (opening through neck into trachea or “wind pipe”) can be achieved through the use of a Blom-Singer StomaSoft Laryngectomy Tube or Blom-Singer Tracheostoma/Adhesive Housing, supplied separately.

INDICATIONS (Reasons to prescribe the device or procedure)

The Blom-Singer HME/HumidiFilter system is designed to provide the laryngectomee with humidified, warmed, and filtered air. The Blom-Singer HME/HumidiFilter system may be used by laryngectomees: esophageal speakers, electrolarynx users, and tracheoesophageal voice prosthesis users.

CONTRAINDICATIONS (Reasons that make it inadvisable to prescribe the particular device or procedure)

The HME/HumidiFilter system should not be placed over a tracheostomal airway if its presence obstructs respiration.

WARNINGS AND PRECAUTIONS

The HME Cartridge and HumidiFilter Foam Filters may be used only once by one patient. They may not be reused. There is a risk of infection if reused. If there are tears, cracks, or structural damage to the device, discontinue use. Care should be exercised when inserting or removing HME devices to avoid aspiration (inhalation) of the voice prosthesis or HME. In the unlikely event that this should occur, you should attempt to cough the device out of the windpipe. Immediately seek medical attention if unsuccessful.

The following complications may occur with devices of this type, but are infrequent. They include:

- Stoma contamination or infection which may require use of appropriate antibiotics.
- Irritation, including skin irritation around the stoma.
- Labored breathing, which may occur if the device is heavily soiled, clogged with mucus, or not changed regularly.

INSTRUCTIONS FOR USE

Please refer to the diagrams located at the front of this instruction manual.

The following instructions are applicable to all HME/HumidiFilter devices. Please refer to additional instructions for your specific device later in this manual.

Device Handling

Handle the devices carefully and with clean hands. Wash hands after handling the devices, as they contain substances that may be irritating to skin and eyes.

Breathing

When you begin using the HME/HumidiFilter System, a slight increase in airflow resistance may be felt. This resistance is a normal function of the devices that may require a period of time to adjust. If breathing through your HME/HumidiFilter System is difficult, remove the device and consult your medical professional for assistance.

Coughing

It is recommended that the HME Cartridge, HumidiFilter Holder and ATSV II with HumidiFilter Cap be removed prior to coughing.

Removal from Housing or Tube

Removal and attachment of any devices worn over the tracheostoma should be done with the user positioned in front of a mirror in a well-lit room. To remove the device, hold down the previously applied tracheostoma housing or tube with one hand while grasping the device (diagram 1C-a) with your other hand. Pull the device carefully until it releases from the housing or tube.

HME Cartridge

The HME Cartridge must be replaced at least once a day to maintain proper function of the HME system.

1. Insert the HME Cartridge into the tracheostoma housing or tube and gently push the HME Cartridge until it snaps into place (diagram 5). A properly inserted HME Cartridge will sit flush and fit snugly inside the housing or tube.
2. Tracheoesophageal speech with a voice prosthesis and HME is produced by pressing down on the EasyTouch speech button (diagram 6). You will feel the speech button completely close. Upon manual release, the EasyTouch speech button returns to its open, breathing position. Note: If the EasyTouch speech button is not completely closed, there may be difficulty voicing or a whistling sound.

Cleaning the HME Cartridge

The HME Cartridge is designed for 24-hour use and is not designed to be cleaned, disassembled/ reassembled, or reused. Mucus may be wiped away with a lint-free cloth as able. If the HME Cartridge should become heavily soiled, remove it and replace it with a new HME Cartridge.

HumidiFilter Holder

The HumidiFilter Holder is designed for repeated use. The HumidiFilter Foam Filter must be replaced daily to maintain proper function of the HumidiFilter system.

HumidiFilter Holder Assembly and Insertion

1. Insert the HumidiFilter foam filter into the holder. The HumidiFilter should lie flat in the holder (diagram 7).
2. Insert the assembled HumidiFilter Holder into the previously applied tracheostoma housing or tube and gently push the holder until it snaps into place. A properly inserted HumidiFilter Holder will fit snugly inside the housing or tube. You should be able to breathe without difficulty through the HumidiFilter.

3. Tracheoesophageal speech with a voice prosthesis is produced by temporarily occluding the opening of the HumidiFilter Holder with a finger or thumb. Note: If the stoma is not completely occluded, there may be difficulty voicing or a whistling sound.

Cleaning the HumidiFilter Holder

Never clean the HumidiFilter Holder while it is in position in the neck. Only clean the HumidiFilter Holder after it has been removed from the tracheostoma housing or tube.

1. Pull the HumidiFilter foam filter out of the Holder with forceps and discard the foam. Retain the HumidiFilter Holder to be cleaned and reused.
2. Clean the HumidiFilter Holder with mild soap and warm water. Rinse with water.
3. Dry with a lint-free cloth or allow to air-dry. Never use facial or toilet tissue to dry the HumidiFilter Holder as particles of lint or fabric may be deposited onto the device that can be inhaled into the airway upon reinsertion.

Sleeping

The HumidiFilter Holder must be removed before sleeping in order to avoid the potential for airway obstruction.

ATSV II HumidiFilter Cap and Foam Filters

The ATSV II HumidiFilter Cap and Foam Filters are intended for use with the Blom-Singer ATSV-II Hands-Free device. The cap is designed for repeated use. The ATSV II HumidiFilter Foam Filter must be replaced daily to maintain proper function of the ATSV II HumidiFilter system.

ATSV II HumidiFilter and Cap Assembly and Insertion

1. Insert the HumidiFilter foam filter (diagram 8a) under the crossbar (diagram 8b) of the ATSV II Faceplate so that one of the slits on the filter is lined up with one end of the crossbar. Pull the HumidiFilter through so that the second slit is lined up with the opposite end of the crossbar.
2. Place the ATSV II HumidiFilter Cap (diagram 8c) over the crossbar (diagram 8b) and rotate counterclockwise to lock it into position.
3. Insert the assembled ATSV II/HumidiFilter Cap into the previously applied tracheostoma housing or tube and gently push the unit until it snaps into place. A properly inserted ATSV II/ HumidiFilter Cap will fit snugly inside the housing (diagram 9). You should be able to breathe without difficulty through the device.

Refer to the Adjustable Tracheostoma Valve Instructions For Use manual for complete instructions on how to properly use the ATSV II for hands-free speech.

Cleaning the HumidiFilter Cap

1. Never clean the HumidiFilter Cap while the ATSV II/HumidiFilter Cap is in position in the neck. Only clean the HumidiFilter Cap after it has been removed from the ATSV II. Remove the ATSV II/ HumidiFilter Cap from the tracheostoma housing or tube.
2. Remove the ATSV II HumidiFilter Cap by rotating the cap clockwise and lifting it off from the faceplate.
3. Pull the HumidiFilter foam filter out with forceps and discard. Retain the ATSV II HumidiFilter Cap for repeated use.
4. Clean with mild soap and warm water. Rinse thoroughly with water.

Dry with a lint-free material or allow to air dry. Never use facial or toilet tissue to dry the HumidiFilter Cap as particles of lint or fabric may be deposited onto the device that can be inhaled into the airway upon reinsertion.

Replacing ATSV II HumidiFilter Foam Filters

Never replace the HumidiFilter while the ATSV II/ HumidiFilter Cap is in position in the neck. Always replace the HumidiFilter after the HumidiFilter Cap has been removed from the ATSV II.

1. Remove the ATSV II/HumidiFilter Cap from the tracheostoma housing or tube.
2. Remove the ATSV II HumidiFilter Cap by rotating the cap clockwise and lifting it off from the diaphragm/faceplate.
3. Pull the HumidiFilter foam filter out with forceps from under the crossbar and discard.
4. Replace with a new HumidiFilter Foam Filter as described above in Assembly.

Sleeping

The ATSV II and HumidiFilter Cap must be removed before sleeping in order to avoid the potential for airway obstruction.

SPECIAL STORAGE AND/OR HANDLING CONDITIONS

Only the foam filter portions of these devices have special storage and/or handling conditions:

- Wash hands after handling the devices, as they contain substances that may be irritating to skin and eyes.
- The packages containing the HME Cartridges and HumidiFilter Foam Filters should be resealed after opening to preserve the effectiveness of the foam.

DISPOSAL INSTRUCTIONS

This device is not biodegradable and can be contaminated when used. Carefully dispose of the device per local guidelines.

ORDERING INFORMATION

USA InHealth Technologies® products may be ordered directly from Freudenberg Medical, LLC, d/b/a "InHealth Technologies".

Telephone: Toll-Free 800.477.5969 or +1.805.684.9337

Fax: Toll-Free 888.371.1530 or +1.805.684.8594

Email: order@inhealth.com

Order Online: www.inhealth.com

Mail: InHealth Technologies 1110 Mark Avenue Carpinteria, CA 93013-2918 USA

INTERNATIONAL

InHealth Technologies products are available through authorized distributors worldwide. For international distributor information, please contact our customer service department.

PRODUCT COMPLAINTS/EU SERIOUS INCIDENTS

If you are dissatisfied with the device or have any questions, please contact

productcomplaints@inhealth.com.

Telephone: +1-800-477-5969

Fax: +1-888-371-1530

Any serious incident that has occurred in relation to the device should be reported to Freudenberg Medical, LLC as noted above and the competent authority of the EU Member State in which the user and/or patient is established.

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SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Batch code; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	EN ISO 15223-1:2016	Use-by date; 5.1.4	Indicates the date after which the medical device is not to be used
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the packaged has been damaged or opened
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	EN ISO 15223-1:2016	Do not re-use; 5.4.2	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure
	ISO 7000:2019	Packaging unit; 2794	To indicate the number of pieces in the package
	ISO 7000:2019	Single patient multiple use; 3706	To indicate that the medical device may be used multiple times (multiple procedures) on a single patient

INHEALTH

TECHNOLOGIES®

UNITED STATES

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
Toll-Free: 800.477.5969
Toll-Free Fax: 888.371.1530

INTERNATIONAL

Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: order@inhealth.com

ORDER ONLINE

www.inhealth.com

Freudenberg Medical, LLC
1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
www.inhealth.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands